


Drugs of Abuse

FOR SELF-TESTING USE

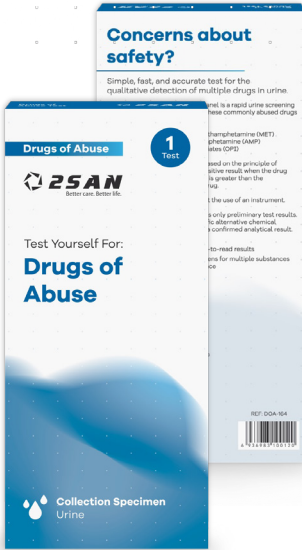

Concerns about safety? Simple, fast, and accurate for the qualitative detection of multiple drugs in urine.

Test yourself for 6 different drugs of abuse in the same test, using a simple at-home urine test with results in 5 minutes.

The 2San Multi-Drug Rapid test gives an indication of the presence of drugs of abuse in your urine. An observed sample or a secondary confirmatory test is recommended depending on the rationale for the test and the drug testing policy in place.

- Simple Urine Test**
- Results in 5 Minutes**
- Easy-to-read results**


Specification

Intended Use	Chromatographic immunoassay for qualitative detection of AMP, COC, MET/mAMP, THC, OPI and BZO in urine	Time to Result	5 minutes
Storage	2-30°C	Shelf Life	2 years
Specimen Type	Urine	Sensitivity	>99.9 %
		Accuracy	>99.9 %
		Specificity	>99.9 %

Components

- 1 Test Midstream
- 1 Package Insert

Procedure

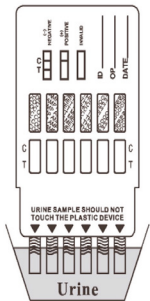

Lists of Drugs

Drug (Identifier)	Cut-Off Levels
Amphetamine (AMP)	100ng/mL
Cocaine (COC)	300ng/mL
Methamphetamine (inc Ecstasy) (MET/mAMP)	100ng/mL
Cannabis (THC)	50ng/mL
Opiates including Heroin (OPI)	200ng/mL
Benzodiazepines (BZO)	300ng/mL

Interpretation

POSITIVE
Two lines appear, the drug is present at concentrations above the cut-off level.

NEGATIVE
Only the control line appears. The drug is not present in concentrations above the cut-off level.

INVALID
The control line does not appear. Repeat the test.

Place the absorbent tip in the midstream in the urine in a cup OR in the urine stream and hold for at least 15 seconds. Read the results at 3 minutes.