



Please read carefully and keep this package insert for future use.

DESCRIPTION:

SYSTANE® Gel Drops Lubricant Eye Gel is suitable for day time and night time application and provides relief from moderate to severe symptoms of dry eye. Its formulation is specifically designed to lubricate and restore the ocular surface, protecting it from further damage.

SYSTANE® Gel Drops Lubricant Eye Gel is a sterile solution containing polyethylene glycol 400, propylene glycol, hydroxypropyl guar, sorbitol, aminomethylpropanol, boric acid, potassium chloride, sodium chloride, edetate disodium, and POLYQUAD® (polyquaternium-1) 0.001% preservative.

USES (Indications)

SYSTANE® Gel Drops Lubricant Eye Gel is a dry eye therapy for the temporary relief of ocular symptoms due to dryness of the eye, such as discomfort, burning and irritation.

CONTRAINDICATIONS (Reasons not to use):

People allergic to any ingredient in **SYSTANE® Gel Drops Lubricant Eye Gel** should not use this product.

DIRECTIONS:

SYSTANE® Gel Drops Lubricant Eye Gel may be used as needed throughout the day and also at night time. May be used to treat dry eye associated with contact lens usage by instilling drops prior to inserting contact lenses and after the removal of contact lenses. Shake well before using. Place drops on the eye and blink.

WARNINGS:

It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use of the product. If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, consult your eye care practitioner as the problem could become more serious.



PRECAUTIONS:

- To avoid contamination, never touch the dropper tip of the container to any surface.
- Replace cap after using.
- Store at room temperature.
- · Keep container tightly closed when not in use.
- Discard any remaining solution three months after first opening.
- Keep out of the reach of children.

HOW SUPPLIED:

SYSTANE® Gel Drops Lubricant Eye Gel, which is sterilized by a combination of steam and filtration, is supplied in plastic bottles in sizes ranging from 3 mL to 10 mL (and multiple bottles thereof).

STERILE A



Alcon Laboratories, Inc. 6201 South Freeway Fort Worth, Texas, USA 76134-2099

EC REP

Alcon Laboratories (UK) Ltd. Frimley Business Park Frimley, Camberley Surrey, GU16 7SR, United Kingdom





www.systane.co.uk



